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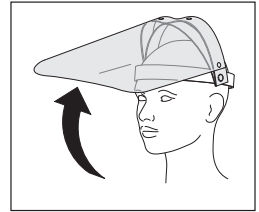
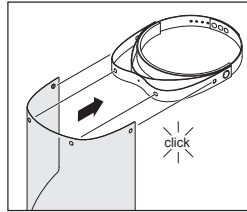
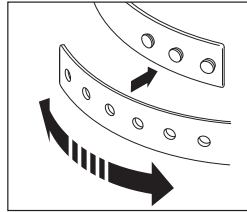
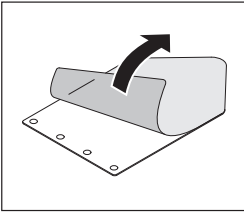
ECS European Certification Service GmbH,  
Hüttfeldstraße 50, 73430 Aalen, Germany  
Notified body no. 1883.

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## EN Operating Instructions

Replacement sheets for the DURABLE eye protective equipment 343310. The sheets for the eye protective equipment can be exchanged. This product is CE-compliant and complies with the PPE/PPE (Personal Protective Equipment) Regulation (EU) 2016/425 category II as well as DIN EN 166:2001, is type-tested and CE labelled.

In order to safely handle this product, the user must have read and understood the operating instructions before using it for the first time.

### Use

This product protects against droplets and splashes of liquids. It reduces the risk of a droplet infection. This eye protective equipment does not provide any protection against mechanical effects. Due to design considerations, complete protection against viruses, bacteria or other biological agents is only provided to a limited extent. Can be combined with face protection masks. Ensure proper disposal after use. Reusing it is up to the user's own discretion.

We recommend using the product only for a maximum period of 2 years. The duration of use depends on different factors, such as usage, cleaning, storage and maintenance. We recommend frequent controls and replacement of the sheet if it has scratches on it.

### Storage

In original packaging, in a dry environment. Storage condition: approx. 18 °C – 23 °C, approx. 55 % humidity.

### Duration of use

Do not use the article for more than 2 years after the date of purchase.

### Cleaning and Disinfection

With clear, lukewarm water and household washing-up liquid (no caustic solutions, acids or solvents). Avoid strong mechanical cleaning. The article can be disinfected with standard, commercially-available disinfectant.

### Risks

This product does not provide any protection against mechanical effects. It does not provide protection against any type of radiation (sunlight, UV, welding glare, IR, among others). The eye protective equipment is only suitable for using at room temperature and is not fire-resistant. If the product is worn over corrective glasses, mechanical effects (for instance impacts) can be transferred, which represent a risk. Do not use the face shield if it is damaged, because this may influence sight and protection. Damaged eye protective equipment must be replaced (sharp edges can cause injuries). Materials that come into contact with the user's skin could trigger allergic reactions in sensitive people.

### Declaration of conformity

You will find the declaration of conformity at:  
[www.durable.eu/3433CE](http://www.durable.eu/3433CE)

### Certification authority

ECS European Certification Service GmbH  
Hüttfeldstraße 50, 73430 Aalen, Germany  
Notified body no. 1883

The article complies with the following regulations of the European Union:  
Regulation (EU) 2016/425 category II

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DURABLE · Hunke & Jochheim GmbH & Co. KG