FotoSan® 630

Manual

CE

FotoSan® 630 ManualPages

English 3 - 12

Dansk.....13 - 22

Deutsch.....23 - 32

Nederlands33 - 42

Notes43

Section	Content
1.....	Technical description and WARNINGS
1.1.....	Technical specifications
1.2.....	Environmental conditions for intended use
2.....	Preparing to use the FotoSan® 630
3.....	Use and program modes
3.1.....	Intelligent Automated Functions
3.2.....	Temperature safety
3.3.....	Battery power saving
3.4.....	Indications
4.....	Cover and tips
5.....	Cleaning
6.....	FotoSan® Agent
7.....	Important treatment principles
8.....	Practical treatment guidelines
9.....	Labelling
10.....	Service and warranty
11.....	Disposal of waste products

1. Technical description

The product is classified as an electronic medical device electric class II according to DS/EN 60601-1: 1998. The product is classified as a class 1 LED product according to EN 60825-1:2001 Ed 1.2.

CMS Dental declares that the product described in this manual complies with the above mentioned standards.

The FotoSan® 630 light consists of a hand piece, a docking station and an external power supply. Disposable covers and tips protect against cross contamination.

WARNINGS

1. Do not expose directly into the eyes, or look directly at the emitted light.
2. Eyewear protection should be used.
3. Medical electronic equipment is sensitive to electromagnetic fields. Portable and mobile high frequency communication devices may interfere with medical devices.
4. In order to maintain proper operation, follow instructions in this manual.
5. Use of the light without a tip may cause overheating of the tissue if the light is used close to the gingiva.
6. Extended use cause tips and cover to be hot and may cause slight burning sensation of mucosa, if touched.
7. Do not reuse covers or tips as this may cause cross contamination.
8. If the light beeps every 2 s when placed in docking station, it is a warning that the charging voltage is 6 V or higher, i.e. a wrong power supply is being used.

9. The FotoSan® agent is a blue dye and it stains clothing.

1.1 Technical specifications

Power supply	
Manufacturer	Dongguan Shilong Fuhua Electronic Co. Ltd.
Model	UEO8WCP-050100SPA
Input power	100–240VAC, 50/60Hz
Output power	5.0VDC, 1.0A
Battery	
Battery type	LiFePO ₄
Nominal voltage	3.2V
Nominal capacity	1.200mA
Re-charging time	2 hours
User capacity	approx. 760 seconds
LED Diode	
Output intensity	2.000-4.000 mW/cm ²
Wavelength	620-640nm (85%), peak 630nm

1.2 Environmental conditions for intended use

Temperature range	+10°C to +40°C
Humidity range	30% to 75%
Atmospheric pressure range	70kPa to 106kPa (0.7 bar to 1.06 bar)
IP class	2X

2. Preparing to use the FotoSan® 630

Connect the power supply under the base of the docking station and to a main electrical outlet. Place the light in the dock and the battery will begin charging. This is indicated by a green light flashing.

3. Use and program modes

The FotoSan® 630 is intended to be used for the photoactivation of the FotoSan® Agent:

10-30 s is usually sufficient as activation in one site. See practical treatment guidelines for more details.

Three program modes

The FotoSan® 630 light has three programs each with different time settings.

To see the program selected, press the two buttons at the same time and hold for 3 s until a beep is heard. The diode will flash the color of the program, green, orange or red.

To change the program, scroll using any button. Stop at the chosen program and wait for the beep. The light will stay in the chosen mode until changed manually.

3.1 Intelligent Automated Functions

The FotoSan® 630 is the first LAD light with this new concept.

The green program mode is the manual mode. Press the large button and the unit will light for 30 s. Press the small button and it will light for 10 s.

In the orange program mode, pressing the large button will give you 20 s of light, a 1 s pause, then again 20 s of light. The small button gives you 20 s of light.

This mode is used for deep pockets where 20 s of activation is needed.

In the red program mode the large button will give you 4 times 10 s activations with 1 s pause between each activation. The small button will give you 2 times 10 s with 1 s pause between each activation. This mode is used for treatments of pockets where 10 s is administered typically for each side of the tooth.

3.2 Temperature safety

The FotoSan® 630 light has a safety cut off to preserve the lifetime of the diode should overheating occur.

When the light gets close to the cut off temperature, you will be warned by intermittent beeping. The light can be used for approximately another 15 s before it cuts off. Overheating is indicated by 3 short beeps followed by fast red flashing.


3.3 Battery power saving Sleep

If the light has not been activated for 5 minutes, or placed in the dock, it will go into sleep mode to conserve energy. Press any button to reactivate it.

Deep sleep

If not placed in the charging dock or activated for 6 hours the light will go into deep sleep mode. It must be placed in the dock for one second to be reactivated.

Intelligent Automated Functions: Program modes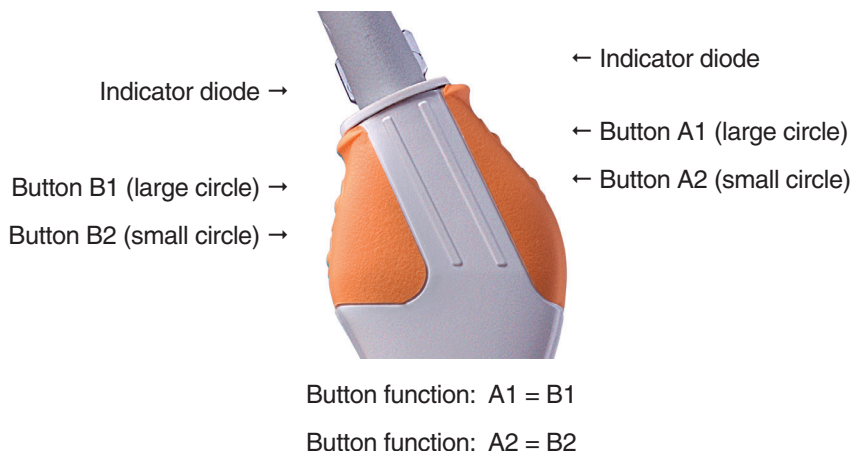

Program change:

● → ● → ● : Press A1+A2 at same time 3 s → scroll → wait 3 s for beep

● A1/B1: 
30s

● A2/B2: 
10s


● A1/B1:  
20s 20s

● A2/B2: 
20s

● A1/B1:    
10s 10s 10s 10s

● A2/B2:  
10s 10s

3.4 Indications: Three-colored diode and sound

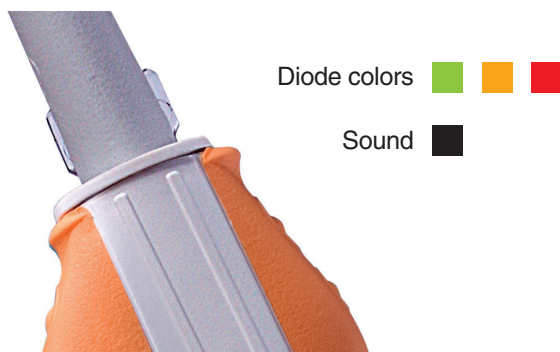Ready		Ready for use
Activated light	 	Sound to indicate start and stop. DANGER! Do not stare into light. It may cause eye damage
Charging battery	 	May be used
Low battery	 	If not used, place in dock
Diode heat warning	 	15 s from cut-off
Diode overheated	 	Needs to cool before it can be activated. DANGER! Tip is hot and may cause light burning to mucosa
Sleep mode		Press any button
Deep sleep mode		Place in dock
Error	 	Reset by pressing two buttons on same side for 3 s. DANGER! No effect of treatment

4. Cover and tips

PERIO
L 23 mm

PERIO
L 15 mm

BLUNT
d 4 mm

BLUNT
d 8 mm

ENDO

The light should only be used with a tip and cover attached. There are five tips for the FotoSan® 630. Use the tip that is best suited for the application. The two perio tips have a diameter of 1 mm. The distal 5 mm have a surface permitting the light to go sideways. The endo tip has a diameter of 0.5 mm.

Attaching and detaching tips

Tips are snapped onto the cover by pressing the two items together. The tips can be moved +/-15 degrees in the longitudinal direction of the cover. To detach the tip, rotate the tip away from the cover and it will pop out.

5. Cleaning

The hand piece and docking station may be wiped with a slightly damp cloth. **CAUTION!** Use only 70% alcohol when cleaning the light and docking station. Make sure NOT to leave fluid in the charger socket, remove excess fluid with a dry clean cloth or tissue. Fluid in the docking socket may cause electrolytic corrosion of the charger.

Make sure NOT to disperse disinfection fluids into the diode. This may damage the diode.

The removeable cover is disposable. The tips are disposable.

6. FotoSan® Agent

The FotoSan® Agent is available in three different viscosities, all with a concentration of active ingredient (toluidine blue O (TBO)) of 0.1 mg/ml. Use the viscosity best suited for the practical application.

The agent is best applied using a blunt (bendable) tip to be attached to the syringe. A 22G or 25G tip is recommended (10 pcs is delivered with syringe box).

Agent	Typical applications
LOW	Endo
MEDIUM	Periodontal pockets
HIGH	Periodontal pockets, surface

7. Important treatment principles

1. You need the agent AND the light to reach the microorganisms for it to be effective.
2. All microorganisms are sensitive to the treatment BUT some more than others - i.e. there is a dose dependency. See literature for details.
3. You cannot treat too much ONLY too little.
4. Difficult accessible infections require longer time than surface infections.
5. You may repeat treatment as needed.
6. You may choose to treat before AND after SRP to reduce bacterial spreading in risk patients.

8. Practical treatment guidelines

Endodontic treatment

Preparation:

Clean and shape the root canal using copious amounts of sodium hypochlorite, citric acid or similar. Dry.

1. Fill the canal with agent LOW, stir to avoid bubbles using a small hand file.
2. Insert the endo light tip until resistance is met and light for minimum 30 s (see table).
3. Rinse and dry.

ENDODONTIC TREATMENT	
Situation	Dosage
If endo tip goes 2/3 into canal	30 s
If endo tip goes 1/2 into canal	2 x 30 s
In case of apical clearing	2 x 30 s AND 10 s with blunt tip on gingiva at level of apex

Periodontal treatment

Preparation:

Perform conventional scaling and root planning.

1. Fill the pocket with agent MEDIUM or HIGH.
2. Treat pocket from inside (perio tip 15 mm or 23 mm) minimum 10 s per surface of tooth, depending of depth. See table.

3. You may find it useful to treat from outside using one of the blunt tips, especially if there is gingivitis.
4. Rinse pocket.

TIP: If the pocket is tight, you may use the endo tip instead.

PERIODONTAL TREATMENT	
Situation	Dosage
PD < 5 mm	10 s per surface
PD < 8 mm	20 s per surface
PD > 8 mm	30 s per surface
Apical periodontitis	Extra 10 s on gingiva at apex position
Gingivitis	10 s per cm ²

9. Labelling

	Operating Instructions
	Caution, see instructions
	Product Serial Number
	Manufacture identification number
	Expiry date
	Medical Electrical Equipment Directive, DS/EN 60601-1:1998 Electrical shock protection, Type B
	Medical Electrical Equipment Directive, DS/EN 60601-1:2006 Electrical double insulation, Class II
	Medical Devices Directive, MDD Directive 93/42/EEC Medical device, Class I
	Waste Electrical and Electronic Equipment Directive, WEEE Directive 2002/96/EC. Dispose product to a recycling central

10. Service and warranty

The FotoSan® 630 light is covered by a 2 year warranty.

CAUTION The FotoSan® 630 light must only be serviced and repaired by CMS Dental, otherwise the warranty will be void.

11. Disposal of waste products

The tips and covers should be disposed as regular waste. The hand piece, docking station and power supply shall be disposed to a recycling central.

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